


## Glide Plate Replacement

1. Remove the Slit Lamp from the table.
2. Mark the placement of the plate on the table in the areas noted in the image below.

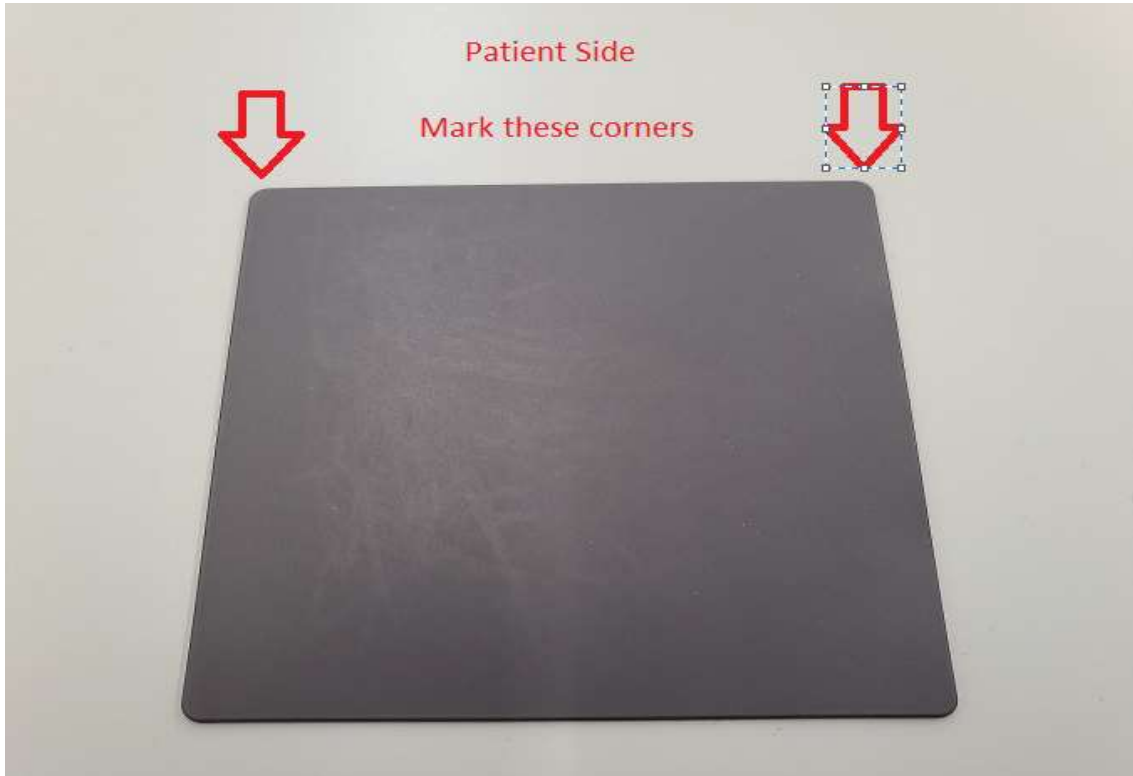

3. With the corners marked, apply heat to the plate using a Heat Gun or Blow Dryer. This will help release the glue. **(Note: Do not allow the heat to concentrate on any particular area for too long it may damage the tables laminate coating.)**
4. While applying heat evenly to the pad, use scrapper or putty knife to separate the pad from the table.
5. Once the plate is lifted remove as much of the glue residue as possible to ensure even adhesion.
6. Remove the covering from the back of the new plate and line it up with the marks made previously.
7. Apply pressure to the plate starting in the center and working out to remove any air bubbles trapped by the glue.