

Octopus 600 Pro Printer Setup

1. Go to **Tools → Preferences**
2. Then under **General** select **Operating Systems Settings**
3. Select the **Tools Tab** in Operating Systems Settings
4. Then select the **Printer Manager button**
5. Click **Add Button**
6. Make sure printer is powered on and USB plugged into Octopus.
7. Select Device Window, **Click** on your printer make and model.
 - a. Lower Right, Connection-USB
 - b. Click **Forward** button
8. Auto install of device driver—usually a few minutes
9. Describe Printer Window
 - a. Verify HP Officejet Pro 6230 is named.
 - b. Click** Apply and Test Print