


Replacing upholstery on 7000, 980, 710, 7000 chairs.


1. Remove the headrest by loosening the single or double headrest mounting screw with a 5/32 Allen wrench. Then slide the headrest up to remove.

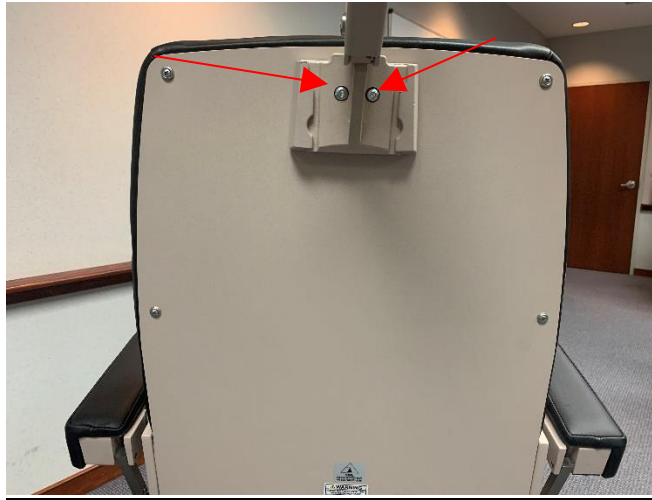

2. Remove headrest bracket by removing the 4 Allen wrench screws using a 5/32 wrench. When removing the bracket, be aware there may be washers between the bracket and the painted back.



3. Remove the 7 screws that hold the painted back onto the chair.
4. Locate the single Phillips head screw in chair back.

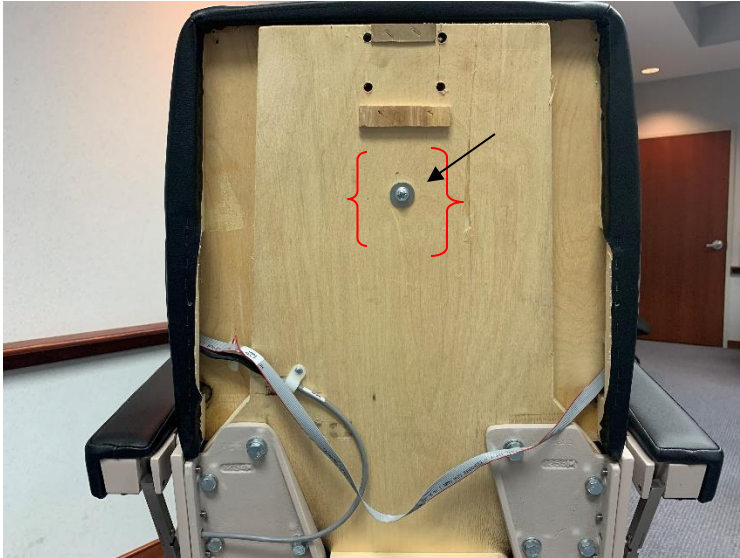

5. After disconnecting wiring to the back switches, then lift the back up and off.
6. There are also a screw located under the seat and apron. Once removed the upholstery can be remove.



Seat Screw

Apron Screw

7. Flip up armrest to remove screws that hold the armrest.
8. Reverse the process to install new upholstery.