

LAMP REPLACEMENT

INSTRUCTION

Dear Valued Customer:

For lamp (bulb) replacement please follow the steps listed below.

1. Disconnect the power supply to the task light

2. Using a small screwdriver, remove the glass retaining ring (3)

3. Take out the glass lens (2)

4. Remove the lamp (with glass reflector) (1)

Caution: Don't touch the lamp, if it is still hot.

5. Insert the new lamp (1)

Be sure that the lamp is properly inserted into the lamp socket

6. Insert the glass lens (2) and the retaining ring (3)

