

### Routine Slit Lamp Cleaning


1. Your slit lamp should be cleaned thoroughly on a regular basis. Weekly is a good rule of thumb. Use a soft cloth to clean the slit lamp housing unit. Never use harsh cleaners on the housing. Do not use sprays. Do not use cloths that drip. Clean optical surfaces after disinfection with a soft cloth.




2. Use a brush to gently wipe down the various surfaces of your slit lamp.



3. Gently clean the lens with a cotton pad, soft brush, or lint-free cloth.

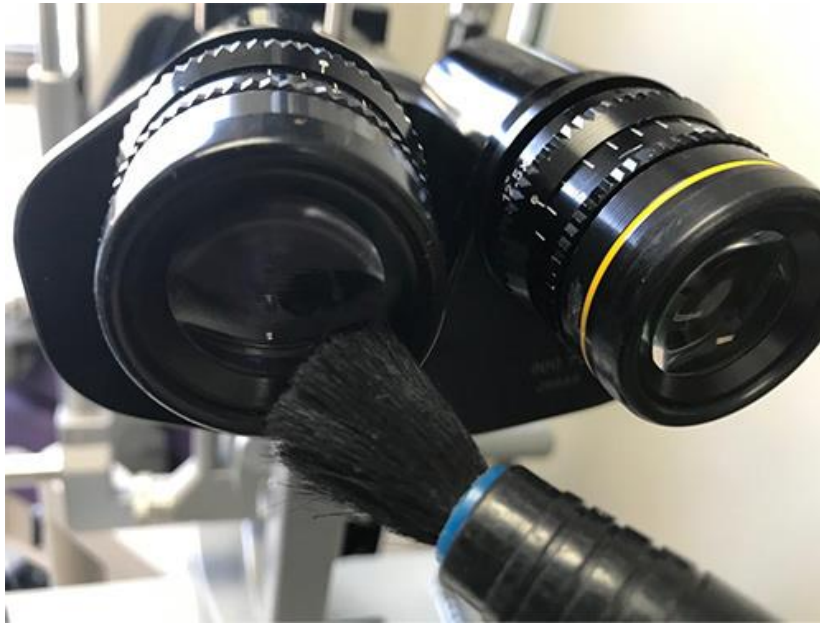

4. Avoid using oils, solutions or water to clean.
5. Clean the mirror with a cotton pad or lint-free cloth.

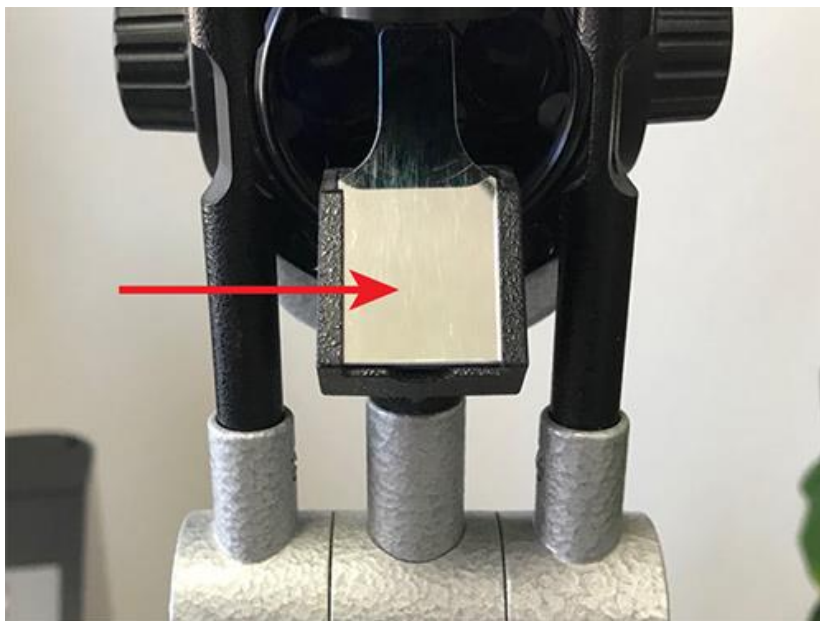

6. The parts of the device, and all applied parts, including the head rest, which are touchable by patient or operator should be disinfected prior to every examination. We recommend using 70% ethanol disinfecting wipes in order to prevent the transmission of infections.



7. Cover your slit lamp at night with a dust cover. If you don't have a dust cover, one can be purchased from the [Haag-Streit USA eShop](#).