



IOL Configuration Quick Guide


1. Open EyeSuite
2. Click the IOL task icon on the left side of the screen (1), then click the Calculation tab (2)
3. Select the template that you would like to modify (3).


Note: You only need to make changes on one eye.

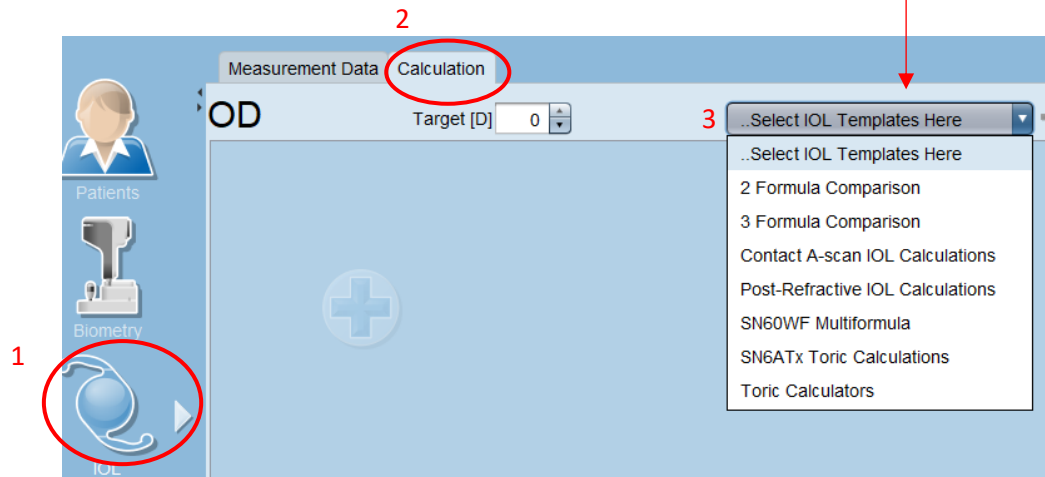

4. Select the lens and formula combinations you wish to see on the template.

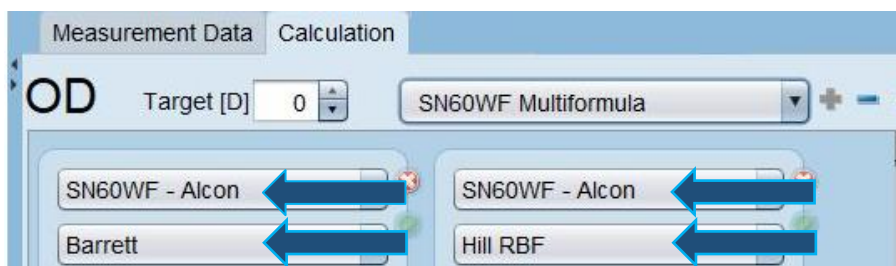

5. Then click on  the "+" symbol next to the template name that now says, "unsaved".
6. A new display will appear, allowing you the option to either (1) type the desired name for the new template **OR** (2) find the existing template name that you would like to override the new changes to, select it, and click OK.

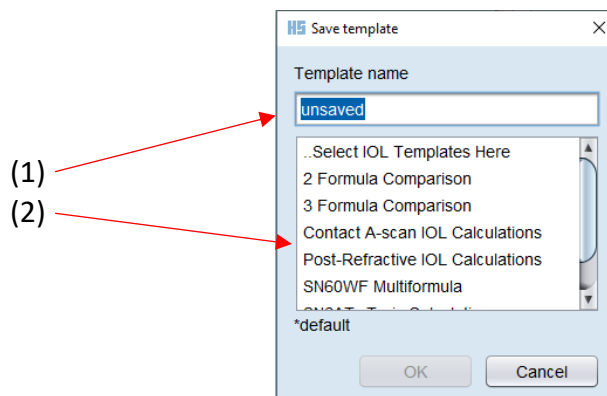

7. This saved template will now be available in the Template Drop-Down menu.